

Addendum to Instructions for Use

Product Name: UniGold HIV

Product Number: 1206506

Instructions for Use: 045-146B Rev 06/07

Dear Valued Customer,

Trinity Biotech Plc is a dedicated partner in the fight against HIV. We are committed to providing quality products and ongoing support, through training and our continual emphasis on customer and technical support.

We would like to re-emphasize the following;

Any reference to a 'line' or a 'line of any intensity' at the test (T) region of the Recombigen device is only deemed a positive test line if it is a 'pink/red line of any intensity'. A line of any other color at this position is not a positive test line and is deemed to be a negative test line.

Similarly for the control line, a valid control line and hence a valid test can only result from the presence of a 'pink/red line of any intensity' at the control region (C)."

Trinity Biotech Plc. is committed to offering quality products and superior customer service. If you have any questions or comments arising from this addendum, please contact us at Trinity Biotech Technical Services (800-325-3424, option 2).

Trinity Biotech USA
2823 Girls Road
Jamestown, NY 14701

545-146B 01/09