



## Rapid Syphilis Test Results in 10 Minutes.



### ***Syphilis Health Check™***

#### **Syphilis Antibody Rapid Immunochromatographic Test**

*Rapid qualitative screening for human TP antibodies in  
whole blood, serum or plasma.*

- First and only rapid syphilis test, with FDA clearance
- Results in 10 minutes
- 98% agreement to other treponemal tests
- Serum, plasma or whole blood
- Whole blood sample collection by finger-stick

Distributed by



Call 800-325-3424 Ext: 7212 or 7225



Each **Syphilis Health Check** kit contains everything required to perform 20 tests

- Syphilis Health Check test devices - 20
- Disposable plastic pipettes - 20
- Diluent in dropper bottle containing saline buffer, detergent and sodium azide ( $\text{NaN}_3 < 0.1\%$ ) - 5ml
- Quick Reference Instructions - 1



**Negative:** 1 colored band in control area

**Positive:** Colored bands in test area AND control area

**Inconclusive:** No distinct color bands in either area

Format	20 tests per kit
Sample	Whole blood, serum or plasma
Collection Method	Finger-stick for whole blood
Accuracy	98% agreement to other treponemal tests
Procedural Steps	2
Time to Result	10 minutes
Storage	Room temperature
CPT Code	86780
Reimbursement	\$18.63

**ORDERING INFORMATION:**

Product	Part Number	Quantity	Storage
Syphilis Health Check™	VSC - 11 - 01	20 Test	4° - 30° C
Syphilis Health Check Control Set	VSC - 11 - 02	1 positive 1 negative control	2° - 8° C

